







BiopSafe makes everyday life easier and safer for doctors, veterinarians, laboratory technicians and nurses – both when working at the hospital, the clinic or "in the field".

# BiopSafe

PRESS FOR BETTER SAFETY

# INNOVATING THE FUTURE

The only constant in the world is change. Axlab Innovation strives to drive that change rather than just following it. We work for solutions that secures our costumers efficiency far in to the future. To do that, we not only have the most advanced technological equipment; we are also equipped for the future because of our innovative thinking and counselling. The results are time savings, a better work environment and last but not least a better service for users and patients now and in the future.



BYGSTUBBEN 12 · DK-2950 VEDBÆK · DENMARK + 45 35 43 18 81



PRESS FOR BETTER SAFETY



## THE NEW ROAD TO A SAFE BIOPSY HANDLING

Sometimes the simple solution is the most difficult to spot. One of the most common problems in connection with biopsy handling is the risk of being exposed to formalin either through touch or inhalation. A risk that doctors, veterinarians, laboratory technicians and nurses are exposed to every day. With BiopSafe the problem is finally solved.

## THE SECRET IS IN THE LID


With BiopSafe there are no difficult formalin bags and wall holders that often leads to formalin waste. The system consists of a small, easy to use container (vial) with formalin capsuled in the lid. When the biopsy is placed at the bottom of the container, you screw the lid on and gently apply pressure with your thumb, allowing the formalin to flow out and cover the biopsy. Everything is done inside the container and no formalin is released in either liquid or vapor form.



The lid is screwed of.



The formalin is released with a press of the thumb.

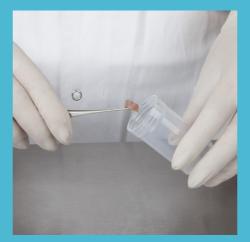

The biopsy is placed in the vial.



The formalin and the biopsy is mixed.



The lid is screwed on.



The biopsy is ready for transportation.

### READY IN SECONDS

Besides from being safe, the method is also fast. BiopSafe is always at hand and the process of placing the biopsy, adding the formalin and making it ready for transportation to the laboratory is done in a matter of seconds. BiopSafe saves valuable time and at the same time ensures that concentration is not broke unnecessarily during operations.

## HERMETICALLY SEALED SYSTEM

As well as causing irritated mucous membranes and allergy, formalin is also a carcinogenic substance. Therefore it is very important to secure a safe working environment when handling formalin. With BiopSafe the handling is done in a hermetically sealed system providing the best protection for the staff. The staff is never exposed to formalin in liquid or vapor form, not even during the process of on and off screwing the lid.