HEINE GAMMA G5[®], G7[®], GP[®], XXL LF[®] Blutdruckmesser

93/42/EWG/CEE/EEC

CE 0297

HEINE

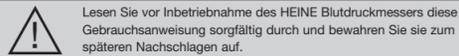
HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49(0)8152/38-0 · Fax +49(0)8152/38-202
E-mail: info@heine.com · www.heine.com
med 0708 1/10.08

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DEUTSCH

HEINE GAMMA G5[®], G7[®], GP[®], XXL LF[®] Blutdruckmesser

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
ENGLISH

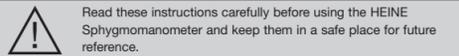HEINE GAMMA G5[®], G7[®], GP[®], XXL LF[®] Sphygmomanometers

Read these instructions carefully before using the HEINE Sphygmomanometer and keep them in a safe place for future reference.

Important note: All components of this sphygmomanometer are latex-free.

Safety Information

Range of applications: HEINE GAMMA G[®] aneroid sphygmomanometers are exclusively designed for measuring blood pressure at the upper arm or thigh on healthy skin.

General safety: Protect the instrument from impact or shocks (do not drop) and protect from moisture and dirt. Avoid damaging the cuff with pointed objects such as needles or scissors.

Do not use the sphygmomanometer in the vicinity of strong magnetic fields.

Safety in use:

Do not take your blood pressure for more than 2 minutes!

Wait for at least 2 minutes before taking a repeat measurement!

Accurate performance of the sphygmomanometer can only be guaranteed if original HEINE G-series latex-free cuffs are used. The use of other cuffs is done so entirely at the liability of the user. The cuff suitable for each individual application is listed in the table.

A rectangular label is attached to the back of the instrument which carries a year-mark. This shows the year in which the instrument should be re-calibrated.

Fitting the cuff

Fit the cuff so that the lower edge is about 2–3 cm above the elbow (or about 5 cm above the knee). Locate the special mark over the artery. The white index mark should be located in the marked index area.

Measurements

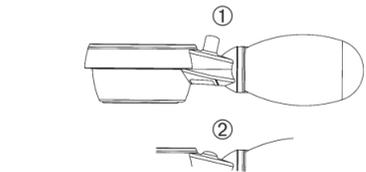
GAMMA G5, G7, XXL LF

Close the air release valve entirely by turning to the right. Pump up the cuff to a value approximately 30 mmHg greater than the expected systolic pressure. Open the valve slowly and regulate the air release to 2–3 mmHg per second. After the measurement, open the valve entirely in order to quick-release the air from the cuff.

GAMMA GP

Press the button to close the valve completely ^①. Pump up the cuff to a value approximately 30 mmHg greater than the expected systolic pressure. Open the valve slowly and regulate the rate of air release to 2–3 mmHg per second. After the measurement, open the valve completely in order to quick-release the air from the cuff ^②.

Please note: It is not possible to pump up the cuff with the button in position ^②.

Changing the position of the spoon (GAMMA G5, G7, GP)

The spoon is normally set for right-handed use. If you wish to inflate the cuff with the left hand, the spoon can be re-positioned for left-handed use:

- Remove the ball from the instrument with a twisting and pulling motion.
- Loosen the two screws near the air intake but do not screw them out completely.

(A special tool is supplied with GAMMA G7 for this purpose).

- Rotate the base of the spoon through 180° and re-tighten the screws.
- Grip the neck of the ball and push it back onto the air intake with a twisting motion. If necessary moisten the air intake to ease fitting.

Cleaning instructions

All parts of the sphygmomanometer can be cleaned with a mild detergent. The liquid must not penetrate the sphygmomanometer. The cuff cover can be hand washed at 30 °C. The bulb must be removed before washing.

Technical data

Range: 0–300 mmHg
Accuracy: ± 3 mmHg

Note: A calibration check should be carried out at the latest every two years by the manufacturer or an authorized service technician.

Applicable standards

EN 1060: "non-invasive sphygmomanometers", Part 1: general requirements, and Part 2: supplementary requirements for mechanical sphygmomanometers

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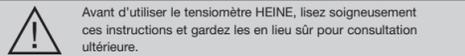
Warranty HEINE GAMMA G5[®], G7[®], GP[®], XXL LF[®]

Instead of the usual 2-year guarantee, we guarantee the HEINE GAMMA G5[®]/GP[®] Sphygmomanometers except for accessories (outer cuff and inflator) for a period of 5 years and the HEINE GAMMA G7[®]/XXL[®] for a period of 10 years, both periods to run from the date of delivery from our factory. For the outer cuff and inflator a 2-year guarantee applies from the date of delivery from the factory.

We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to defects which occur because of wear and tear, incorrect use, the use of non-original HEINE parts or accessories other than of original HEINE manufacture and modifications or repairs carried out by repairers who are not authorised by us. Further claims are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.