Thunder Print Sdn. Bhd.(374290-X)
6463, Lorong Ayam Didik 2, Taman Ria Jaya Light Industrial Park,
08000 Sg. Petani, Kedah Darul Aman, Tel : 04-4418 005 Fax : 04-4424 005

CUSTOMER : RICHTER RUBBER SDN BHD	MATERIAL : SMPO 070	COLOUR : NBLK0001 + NBLK0001	Attn : <u>CATHRINE</u>
DESCRIPTION : LEAFLET HEINZ HERENZ PROBECOVER-CE1639 - HEI000003-LF	FINISHING : FOLDING		From : <u>SELLA</u>
CLOSE SIZE : 148.5MM (W) X 210MM (H)	MOULD NO : -		I hereby verify & confirm that this draft is the final copy. Any subsequent changes of the design and content of the end product will not be borne by THUNDER PRINT SDN. BHD.
OPEN SIZE : 297MM (W) X 210MM (H)	REMARK : _____		Customer Signature & Date

12.1.2024 - 6TH ARTWORK AMEND

PROBE COVERS INSTRUCTION FOR USE

Product Description: Probe covers are made from high quality natural high ammonia (HA) latex or natural vytex (VY) latex. Probe covers made from vytex latex are hypoallergenic with low protein content. The probe cover is used as hygiene protection for ultrasound transducer / probe during transvaginal and transrectal ultrasound examination. The cover allows use of the transducer / probe in scanning while helping to prevent cross contamination.

Directions to use: The probe cover intended for use by or as directed by a licensed medical practitioner during medical ultrasound imaging procedure. If the Probe Covers are sealed in composite foil, please open carefully at the serrated edge of the sealed package. Do not use sharp objects like scissors which may damage the Probe Covers.

Place the Probe Cover over the diagnostic ultrasound transducer / probe before it is used for ultrasound examination. Please observe the right side of the probe cover. Unroll it down with care over the probe of ultrasound scanner with 2 fingers. Do not overstretch the latex film. An appropriate amount of gel placed on transducer / inside the probe cover to prevent poor imaging.

Precautions: The latex probe cover is for single use only. If the probe cover is used more than once, it increases the risk of it tearing and damage and the infection of patient. Never wash it. Do not use, if the package or probe cover is damaged.

To lubricate it, use only water-based gels. Do not use oil-based lubricants; they can damage the latex probe cover.

Storage: Keep the probe cover in a cool dry place away from direct sunlight and heat. Before use, check the expiration date on the package. If the date has passed or if the packaging is damaged, do not use probe covers.

Advice: This product contains natural rubber latex, which may cause allergic reactions including anaphylactic shock. If the rubber material is sticky / brittle / torn or obviously damaged, do not use it.

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rrt Vertrieb und Service GmbH, Heinrich Kümmel Str. 3, 30169 Hannover, Germany.
Heinz Herenz Medizinalbedarf GmbH